

Parafriicta[®]

Low Friction Technology™

Low Friction Fabric

Single, semi-fitted Mobility assist SHEET

With Parafriicta[®] low friction fabric panel
to ease sliding in and out of bed
and turning over whilst in bed

PFFMAS-001



Low Friction Fabric

Parafri^{ct}a[®]

Low Friction Technology™


Single, semi-fitted MOBILITY ASSIST SHEET

The Parafri^{ct}a[®] Mobility Assist sheet is designed to aid users with limited mobility in sliding in and out of bed and also in turning whilst in bed.

- Is particularly useful for patients with neuromuscular (e.g. Parkinsons) and arthritic conditions
- can be used in a clinical or home setting
- can be washed and reused

Instructions for use

- The sheet is designed to be fitted over a standard UK single mattress 90cm wide x 190cm long, including static, pressure relieving mattresses and mattresses that are raised up at the head of the bed. The Parafri^{ct}a[®] label indicates the top side of the sheet.
- The sheet should be fitted over the corners of the mattress at the head of the bed and tucked in at the foot of the bed (see diagram). The Parafri^{ct}a[®] label indicates the top side of the sheet.
- The central panel of Parafri^{ct}a[®] fabric is 66cm wide x 132cm long and should be positioned under the trunk of the user (see diagram)
- The feet should rest on the polycotton part of the sheet to allow the user to gain purchase and to turn (see picture)
- Consider using a moisture absorbing pad under the sheet to draw away liquids
- Change the sheet if there is any soiling or according to clinical judgement

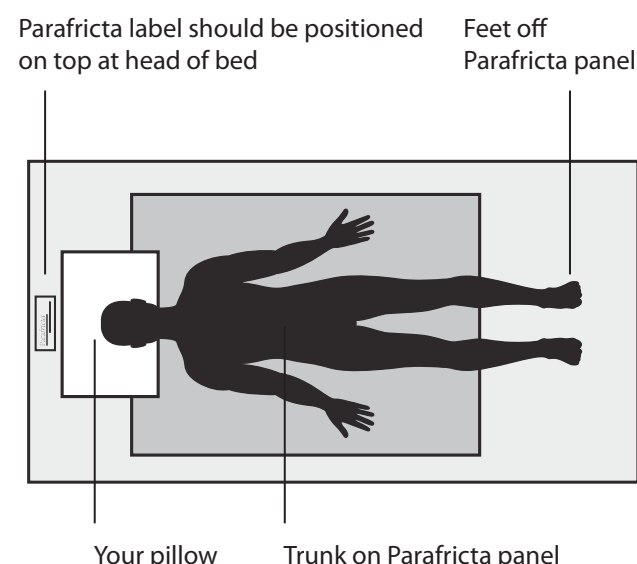

Washing instructions

- Parafri^{ct}a[®] sheets should be washed in a hot (60°C), synthetics cycle in a domestic washing machine. They may also be tumble dried on a low synthetic cycle. See the laundry instructions on the care label.
- In a hospital or commercial laundry the sheets may be washed as for bed linen on a cycle that includes 70°C for up to 10 minutes or on a 95°C cycle for maximum sanitisation.
- Dispose of if there is heavy soiling not removed by washing or if there are any signs of damage to the Parafri^{ct}a[®] fabric side

Place barcode label here

This class I medical device is manufactured in the European Union by

Parafri^{ct}a[®]

Low Friction Technology™



APA Parafri^{ct}a Ltd

I-Lab Bedford, Stannard Way, Bedford, MK44 3RZ UK

Tel 0800 0431412 or +44 1234 834525

info@parafri^{ct}a.com

www.parafri^{ct}a.com

© 2016 APA Parafri^{ct}a Ltd



Patent nos. GB2403149, EP1641420, CN1812759, JP5117721, AU2004251487 and patents pend-ing
"Parafri^{ct}a" is a registered trademark of APA Parafri^{ct}a Ltd