


Parafricta®

Low Friction Technology™

Low Friction Fabric

NEONATAL SHEET

For use in incubators. Parafricta® fabric to protect very fragile skin in neonates from damage caused by **friction** and associated **shear**.

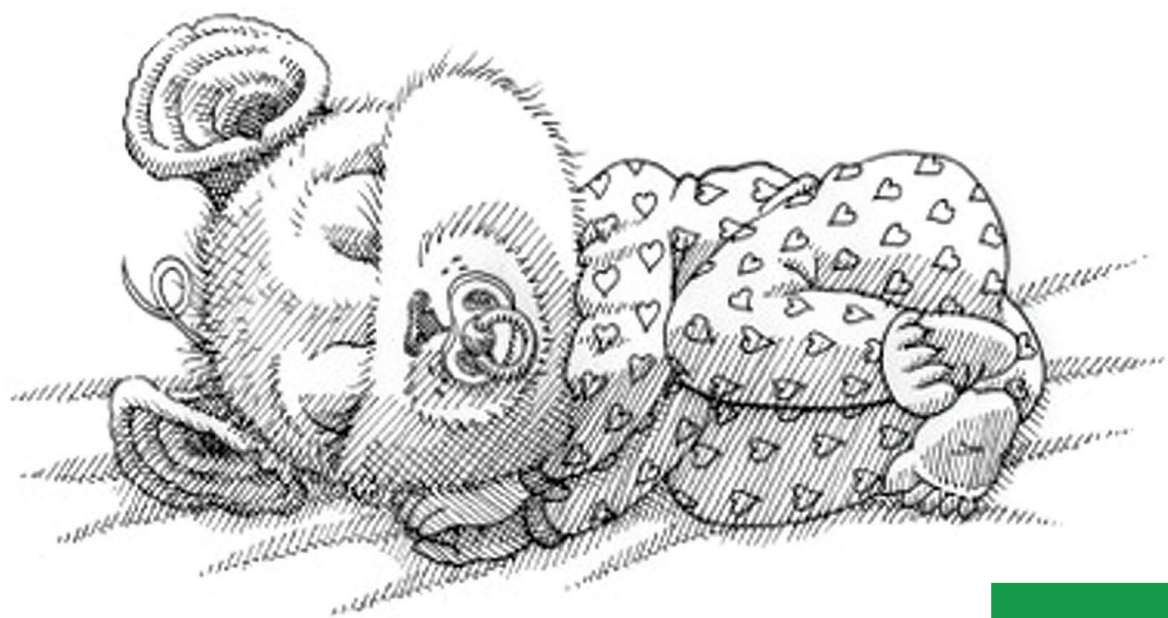

PFFNBS-001



NICU NEONATAL SHEET

The Parafri^{ct}a[®] NICU Neonatal Sheet (74cm x wide x 106cm long) is designed to protect very fragile skin of neonates against the damage caused by **friction** and associated **shear**.

- Helps to prevent, and aid in the healing of, pressure ulcers and friction lesions.
- Can be washed and reused.

Instructions for use

- **IMPORTANT—the Neonatal Sheet must be washed before first use. SINGLE PATIENT USE ONLY.**
- The sheet should be used to cover the mattress in an incubator and tucked-in to keep it flat.
- Baby's skin that is at risk of friction damage should be in contact with the sheet.
- Change the sheet if there is any soiling or according to clinical judgement.

Washing instructions

- The neonatal sheet may be washed in a hospital or commercial laundry on a cycle that includes 70°C for up to 10 minutes or on a 95°C cycle for maximum sanitisation.
- It may also be tumble dried on a low synthetic cycle.
- Dispose of if there is heavy soiling not removed by washing or if there are any signs of damage to the Parafri^{ct}a[®] fabric.

Place barcode label here

This class I medical device is manufactured in the European Union by

Parafri^{ct}a[®]

Low Friction Technology™

"Little Tew" is a licensed, copyrighted image of Crumble Cottage Ltd © 2015



APA Parafri^{ct}a Ltd

I-Lab Bedford, Stannard Way,
Bedford, MK44 3RZ UK

Tel 0800 0431412 or +44 1234 834525

info@parafri^{ct}a.com

www.parafri^{ct}a.com

© 2016 APA Parafri^{ct}a Ltd



Patent nos. GB2403149, EP1641420, CN1812759, JP5117721, AU2004251487 and patents pend-ing
"Parafri^{ct}a" is a registered trademark of APA Parafri^{ct}a Ltd